## Functional Measures of Dexterous Fingertip Prosthesis















Carlos Martinez-Luna | Todd Roberts | Carlton King | Thane Hunt | Brianna Rozell | Todd Farrell, Ph.D. Liberating Technologies, Inc., a College Park Company International Society for Prosthetics and Orthotics 17<sup>th</sup> World Congress 7 October 2019



## **BACKGROUND**

- Existing terminal devices offer trade-offs
  - Form
  - Function
- Some users swap between multiple devices per task
- Aim:
  - Design a solution within a single terminal device





# POINTDEXTER

- Prosthetic fingertip gripper
- Add-on for existing multi-articulating hands
- Combines the best aspects of grippers and hands
- Improve small object manipulation of multi-articulating hands
- Patent-pending design







# DEVICE

- Fully mechanical design
- Uses existing finger motor
- Mechanical switch between finger and dexterous modes
- Controlled in Trigger Grip









## STUDY DESIGN

- Quantify improvements in small object grasping ability using objective functional outcomes measures
- Increase prosthesis use to reduce overuse injuries in the intact limb from this relatively young population





## TEST CONDITIONS

- Confirm that Pointdexter does not interfere with standard hand function
- Compare to highly-functional split-hook
- Conditions:
  - Unmodified Bebionic Hand
  - Bebionic Hand w/ Pointdexter
  - Motion Control Split-hook ETD
- Order of the test conditions was randomized





## FUNCTIONAL MEASURE

- Several hand functions tests evaluated
- Jebsen-Taylor Test of Hand Function
  - Gross and dexterous grasping
  - Subtasks
    - A Spoon Feeding
    - B Card Turning
    - C Empty Cans
    - D Full Cans
    - E Stacking Checkers
    - F Coins and Bottle Caps
    - G Writing
- Three repetitions of tasks





## <u>METHODOLOGY</u>

- Allowed to select the desired grasp pattern in the Hand condition
- Choose whether or not to use to gripper in Pointdexter condition
  - Pointdexter <u>selected</u> for
    - coin/caps (F), card turning (B), stacking checkers (E), and lifting an empty can (C)
  - Pointdexter not selected
    - for holding a pen (G), holding a spoon for feeding (A), and picking up a full can (D)





## **RESEARCH PARTICIPANTS**

#### • n = 4

- 2 experienced trans-radial users of myoelectric multi-articulating hand prostheses
- 2 able-bodied using a prosthesis simulator
- Means and SDs of scores were comparable so the results were pooled
- Informed consent received
- Participants practiced until comfortable with each device to reduce learning effects







# <u>RESULTS</u>

- Jebsen-Taylor tasks were scored by completion time
- Lower time is faster
- Full statistical analysis not conducted due to small n, but several trends were clear
  - Pointdexter showed a 37% improvement over the unmodified hand for small object manipulation
  - Almost as fast as split-hook



## Average and 95% CI Completion Time for



# **DISCUSSION**

- Pointdexter performed similar or better on all other tasks, except card turning
  - Alignment of the small jaw took more precise placement
  - Pointdexter appealing for tasks that it may not be best suited for
  - Real world practice would help to identify these tasks
- Subject feedback for Pointdexter improvements
  - Greater grip strength
  - Wider opening
  - Fit different size hands







## **CONCLUSION**



Pointdexter functioned as an upgrade to existing multi-articulating hands with an improved ability to manipulate small objects while retaining the form and functions of the hand as a whole.



## <u>ACKNOWLEDGEMENT</u>

Research reported in this presentation was supported by the Eunice Kennedy Shriver National Institute of Child Health and Human Development of the National Institutes of Health under Award Number R43HD090800. The content is solely the responsibility of the authors and does not necessarily represent the official views of the National Institutes of Health.



Eunice Kennedy Shriver National Institute of Child Health and Human Development





# **Collegepark**

### TECHNOLOGY for the HUMAN RACE



Carlos Martinez-Luna carlos.martinez@liberatingtech.com +1 (774) 233-0879